EL EN

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MOBIAK[®]
MEDICAL • GASES • FIREFIGHTING est. 1977

CE 9

PRODUCT DESCRIPTION

En

The walker is suitable for users with limited mobility.

It facilitates their movement and at the same time provides support and confidence in their every step.

Ideal for indoor and outdoor use.

PRODUCT USE

Because of its wheels and versatility, the rollator stroller is designed for people who can walk, but need some help with balance and stability. It can maintain the user's energy and allow for a faster form of walking by assisting regular walking.

INSTALLATION

"Unfolding" the Rollator

Open the safety lock and pull out the operating handles (see figure 1).
 Press the sidebars until you hear a small click in the fully folded position (see Figure 2).
 Before using the roller, make sure that the folding mechanism is secured in place.


Rollator height adjustment

Lift the fuse and push or pull the handle to the correct height.
 Push down the cover, the handle will now be locked to the correct height. (See Figure 3)

Bag Installation / Uninstalling

To place the bag strap on the bag, place the hook at the front end of the seat bar outward (see Figures 4 & 5). Once it is secured, secure it with Velcro straps (Figure 5a).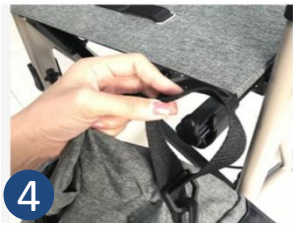

Braking / stopping

Pull both brake levers upwards to apply the brakes while the rollator is moving (Figure 6). Always brake slowly and steadily. Push the two brake levers down to lock while the rollator is not in motion (Figure 7).

Pull up the brake levers to release the brakes.

Frequent brakes are affected by the friction of the wheels and wear. Regularly check the brakes.

Brake adjustment

To adjust the brake, pull up the brake cable (Figure 8) adjust the brake by pushing the nut up or down (See Figure 9)

Sitting on the rollator

Place the rollator on a flat, firm surface. Close the brakes (parking position).

The rollator must stand absolutely firm.

Sit straight with a straight back and do not lean back. (Figure 10).

--CAUTION--

DON'T MOVE THE ROLLATOR WHILE SEATED.

"Folding" and storing the rollator

To fold it, pull the rollator seat upwards from the seat handle as shown. (Figure 11). The rollator can now be fully folded and secured using the safety lock as shown (Figure 12).

Small barriers foot adjusted

In case you need to cross your rollator over small obstacles, such as sidewalks and curbs, it is recommended to use the built-in access barrier (Figure 13). Just press the lever and with a gentle movement back to the handles, your rollator will lift the front wheels for easy handling over obstacles.

Cane / crutch support stand

The elegant cane / crutch mount consists of two parts, a hook at the top and a folding base at the bottom, and is used to place a walking stick or crutch by the user and carry it during walking.

Glass / bottle holder

The rollator comes with a cup/ bottle holder, which is ideal for carrying liquids into a cup or bottle.

DECLARATION OF CONFORMITY

We declare the conformity of the above product to the provisions of the Medical Equipment Directive 93/42/EEC and the Greek National Legislation ΔΥ8δ/Γ.Π.οικ.130648. ΦΕΚ 2198Β/02-10-2009

Each product in this statement meets the technical specifications contained in its technical documentation if the appropriate maintenance will be provided and the product will be used according to the "instruction of use" then the product is safe and effective and does not endanger the health safety of patient's, user and or third parties.

This declaration is issued on the basis of a certificate of registration in the Manufacturer Register of Medical Devices of the Competent Authority (EOF). The product belongs in Class I, Classification rule 1 & Corresponds to the Medical Device Directive: 93/42/EEC

SPECIFICATIONS

Maximum adjustable height	105 cm
Minimum adjustable height	88 cm
Wheel to wheel length	62,5 cm
Wheel to wheel width (unfolded)	61 cm
Wheel to wheel width (folded)	23 cm
Seat height from the ground	60 cm
Front wheels	Soft EVA 20 cm diameter material rotatable 360°
Back wheels	Soft EVA material diameter 18 cm
Frame material	Aluminum
Maximum patient weight	136 Kg
Maximum bag carriage weight	5 Kg
Product weight	6,5 Kg

IMPORTANT (!)

PRECAUTIONS

- Use only as a walking aid!
- Use only on uni-form and compact surfaces, pavements and sidewalks.
- Maximum user weight: 136 kg.**
- Regularly check the screws and fasteners on the Rollator and tighten if necessary.
- Only use a bag for the carriage of small items and do not overload it. **Maximum load: 5 kg.**
- Do not expose the product to temperatures above 40°C for a long time, as this may damage the product and cause injury to the user.
- Make sure that the swivel wheel is fully open, the seat is down, and the folding bar under the seat is locked in the lower position.
- Make sure that both handles are fully tight and have equal heights.
Test its stability before applying the full weight patient to the handles.
- Check the brake lever before using the rollator. When the wheels are locked, the locking bar on the rear wheels must prevent the wheels from rotating. When the fuses are released, the wheels must spin freely.
- Always lock the brake** lever before seating and do not use the seat when the product is tilted or on uneven ground.
- Replace tires or brakes when they are severely worn or damaged.
- DON'T MOVE THE ROLLATOR WHEN SEATED.** This is only a walking aid and should not be used as a transport device.

MAINTENANCE

Mobiak SA products are carefully monitored and provided with a mark when marketed. In case of repair, you only require genuine spare parts and accessories.

CLEANING AND DISINFECTION

Use only a damp cloth and mild soap on surfaces where they need cleaning. Then dry thoroughly before using the product.

If you have to disinfect the product, use a common, mild disinfectant.

Never use acids, alkalis or solvents such as acetone or dissolving fluids.

IMPORTANT (!)

In case of failure or malfunction of your device, please contact your authorized and trained dealer.

Use of the following device is prohibited for purposes other than those specified in this manual

MOBIAK SA disclaims any responsibility for damage resulting from improper use of the device or use other than that set forth in this manual.

MOBIAK SA reserves the right to make changes to the device and the following manual without notice in order to improve its features.

Environmental Protection

If you find out that your device needs replacement and it is no longer useful for you, consider protecting the environment:

- 1) Do not dispose of your device with other municipal waste (this is also the meaning of the recycling symbol listed).
- 2) Contact your local authority to tell you where your appliance is to be recycled.
- 3) Having your junk device in the right place recycling helps protect the environment as well as the material handling of your device.

WARRANTY

The product is guaranteed a two-year (2) warranty on the rollator frame from the date of sale of the product.

The warranty covers factory defects & **DOES NOT** cover damages resulting from misuse, poor maintenance, modification, abuse or non-compliance with the product's operating instructions. Also **DOES NOT** cover worn out parts such as tires, brakes, fabric and that can wear out during use or over time.

Damages or defects caused by: natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty.

Warranty Form

Buyers Data

Name:

Date of purchase: S/N.....

LOT:..... Phone:

Resellers Data

Name :
.....
.....

Address:.....
.....

Phone:

Reseller Stamp

MOBIAK S.A
Kathiana Akrotiriou
Chania-Crete, Greece
+30-28210-63222 www.mobiakcare.com