EL EN

MOBIAK®

MEDICAL • GASES • FIREFIGHTING est. 1977

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**MOBIAK**[®]
MEDICAL • GASES • FIREFIGHTING est. 1977

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PRODUCT DESCRIPTION

EN

Anti-decubitus mattresses are specifically designed to treat and help bedsores in bedridden patients. They can be used for hospital or home use. Evenly distribute the pressure on the body to prevent edema.

The pump switches the air pressure in the mattress, creating a sense of light massage and activate the immobilized due sprawling cells.


This product is indicated and suggested for patients who are likely to spend more than 15 hours a day in bed and for patients with medium to high risk of developing ulcers & bedsores (Stage II to III)

OPERATION USE

For the stripped mattress

- Place the air mattress over the bed mattress, having first stretch the bottom sheet, making sure the air inlet is at the side of the legs.
- Secure it with fixing straps below the mattress
- Place over the air mattress a thin breathable fabric, so it does not come in direct contact with the patient's skin.
- Attach the mattress to the pump using the appropriate tubes in the package, when you hear the "click", it means that they are properly fitted & secured.
- Connect the pump to the power supply, press the **ON / OFF** switch, and turn the switch to the maximum to inflate the airfoil BEFORE used by the patient
- When the airway is fully inflated you can adjust the pressure range, depending on the user requirements, using the switch

For Air Pump: DESCRIPTION DISPLAY / FUNCTION

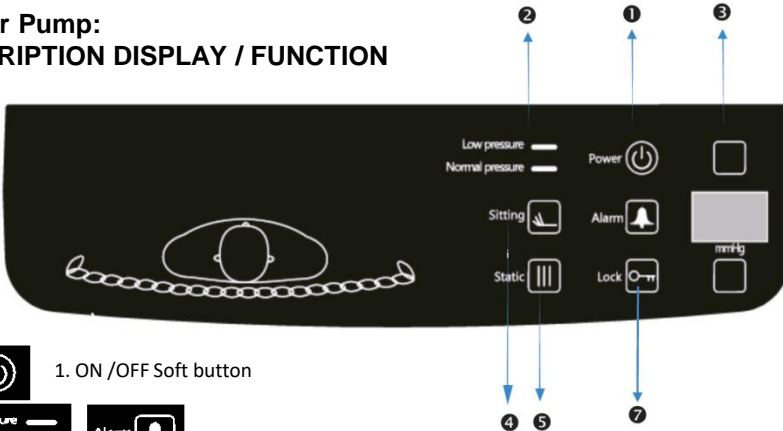
1. ON /OFF Soft button

2. LOW PRESSURE

If the air pressure is lower than the specified level, an alarm can be triggered where it can be recalled

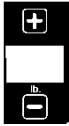3. AUTOMATIC PRESSURE CONTROL

Display of pressure range adjustment in kg of patient (30-180 kg.).
 Hold down the key to jump fast, each step is 10kg.

4. SEAT OPERATION

The display shows the letter "L"
 It offers extra support to patients when they are seated.

5. STATIC OPERATION

Both zones inflate together, no pressure switching, LED display shows "- -" for 30 minutes and then switch back to Alternating Pressure.
 Use only for nursing treatment.

4. In the event of a power failure, an alarm will sound, press the "ALARM" key to mute until the power returns.


5. By pressing it, it locks all functions of the device.

To switch it off or on, hold it down for 5 seconds continuously.

SPARE PARTS

REF. NUMBER	DESCRIPTION
0806289	Anti-Decubitus STRIPE Mattress 5"+3" FOAM Mattress
0806290	Pump for Anti-Decubitus STRIPE Mattress 5"+3" FOAM Mattress


Front

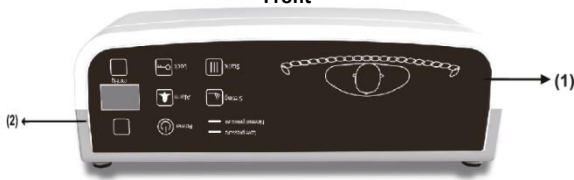
- 1. Control Panel
- 2. Quick Connection female


- 3. Quick connection male
- 4. Fuse
- 5. Filter
- 6. Filter cover
- 7. Bed hooks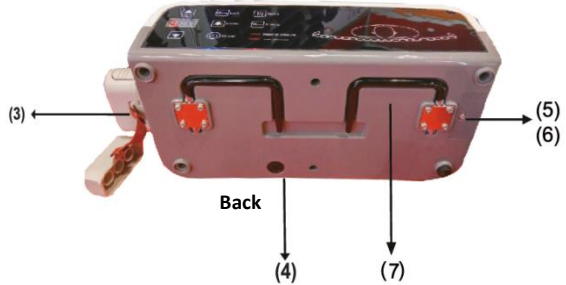

INSTALLATION

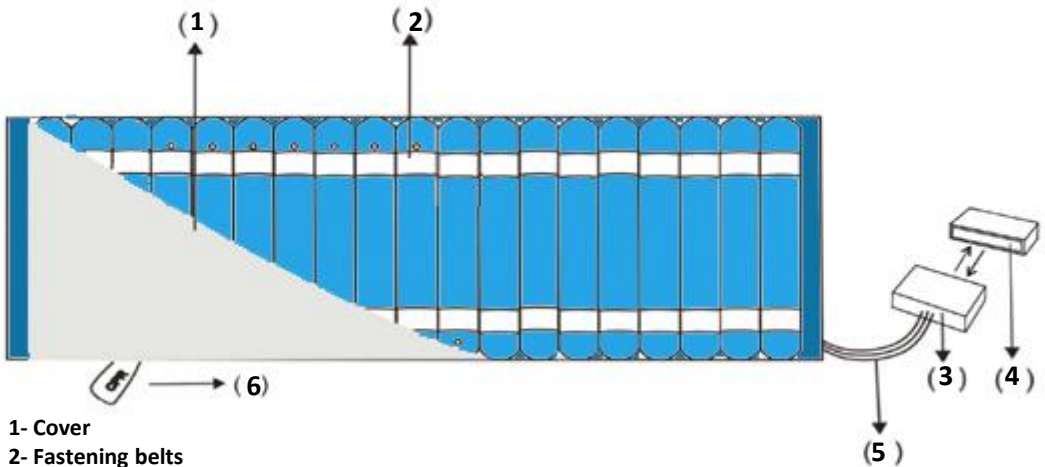
- 1- Cover
- 2- Fastening belts
- 3- Quick connection male
- 4- Quick connection cover
- 5- Air supply pipes
- 6- CPR valve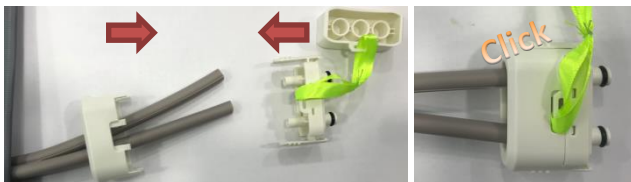

- Place the mattress directly on the bed frame with the tubes going to the pump facing downwards.
- Hang the air cushion pump on the bed where the legs end up or place it on a firm flat surface.
- Connect the quick links as shown in the pictures left.

- Connect the male part of the quick connection to the female part where it is located on the pump.
- Next, check the air supply hoses so that they are not beneath the mattress or that they are not folded to a degree of air pass difficulty.
- After the final check, connect the power supply to the wall power supply.
- Turn on the device.
- We wait about 12 minutes for the airbag to reach its maximum air capacity. Once this is done, the Normal Pressure light will turn green. The Low Pressure lamp will blink for the entire duration of this inflation.

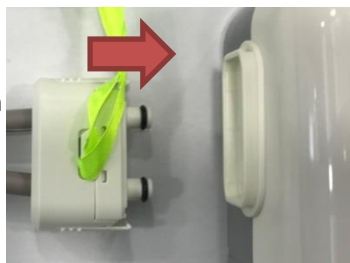NOTICE

If Cardio-Pulmonary Resuscitation (CPR) is required in the patient, immediately pull the valve (CPR) to release air directly from the mattress.

The valve (CPR) is located on the right side at the top of the mattress.

Also remove the connection from the pump for an even faster deflation of the mattress.

SPECIFICATIONS

PUMP	Operating Voltage (V) -Frequency (Hz)	220 VAC/ 50HZ
	Air outlet	≥ 7 liters /min
	Pressure range	30mmHg – 60mmHg
	Rotation cycle timing	12 min
	Power	≤ 7 W ± 10%
	Noise	≤ 30 dB ± 10%
	Dimension (MxΠxY)	27,5 x 13,5 x 9 cm
	Pump weight	2 Kg
	Maximum inflation time	Approximately 12 minutes
	User interface type	All Keys are Touch, only the On / Off button is soft touch
MATTRESS	Max patient weight	155 Kg
	Color	Blue
	Material	Nylon TPU & Stretchable PU (cover)
	Air Tube Height (inflated)	20 cm
	Mattress Height	0,22 mm
	Dimensions	200 x 90 cm (18 cells)
	Height of foam mattress	8 cm
	Connection tube length	150cm
	Weight	10 Kg

STORAGE

- Switch off the device, unplug it from the power supply and disconnect the air-mattress from the pump.
- Place the air-mattress on a flat surface and roll or fold from the head end towards the foot end.
- Allow air slowly escape the mattress so we will reduce storage space
- Close the protective cap cover on the quick connector.
- Place air-mattress & the pump in the carrying bag (if available) and store it in a moisture-free place.

MAINTENANCE

To clean the mattress use a soft cloth or sponge & water with mild soap and dry well.

ATTENTION:

- Do not use acidic & caustic dilutions.
- Clean the upholstery after each daily use.

We recommend that you clean the upholstery with a mild detergent to protect it from oil stains and dust. A common practice is to use a spray bottle filled with a mixture of anti-bacterial soap and water. Spray the surface and then wipe with a clean towel. The best solution is to be cleaned with warm water mixed with a mild soap and then thoroughly dried.

- If you need to save the mattress, empty of air by folding it
- Keep or store the pump in a clean, dry, well ventilated environment.
- Before cleaning the pump, disconnect from the electrical supply.
- Clean the pump with a dry cloth.

ATTENTION:

Do not bring the pump into contact with liquids.

IMPORTANT (!)

CAUTIONS

- Do not use, while bathing the patient on the bed.
- Protect the product by oils, lotions and creams, by covering it with the cotton wool fabric, or a bath towel.
- Do not use abrasive alcohol-based cleaning when the mattress. This could damage the material of the layer.
- Do not smoke & do not use any flame near the mattress.
- Do not have the mattress in contact with direct heat sources.
- Protect the layer from scratches, cuts and punctures.
- Do not leave the product run on, If not using it.
- Do not disconnect the hoses while the pump is running.
- Do not use sharp objects near the mattress.
- Do not exceed the maximum load (kg) recommended by the manufacturer

IMPORTANT (!)

In case of damage or malfunction of the pump, please contact your authorized and trained dealer.

CAUTION! (!)

Do not use the product for a purpose not indicated in this manual

Only qualified personnel or trained for the treatment and prevention of bedsore, can operate with the device

MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration of the product.

MOBIAK S.A reserves the right to change the information contained in this document without previous notice.

EC Declaration of Conformity

We declare the conformity of the above product to the provisions of the Medical Equipment Directive 93/42/EEC and the Greek National Legislation ΔΥ8δ/Γ.Π.οικ.130648. ΦΕΚ 2198B/02-10-2009

The product belongs in Class I, Classification rule 1 & Corresponds to the Medical Device Directive: 93/42/EEC

Each product in this statement meets the technical specifications contained in its technical documentation

If the appropriate maintenance will be provided and the product will be used according the “instruction of use” then the product is safe and effective and does not endanger the health safety of patient’s, user and or third parties,

This statement is issued based on the certificate of registration in the Register of Manufacturers of Medical Devices in National Organization for Medicine

Environmental Protection

If your device needs replacement, or don’t use it, consider the environment:

- 1) Do not dispose of your device along with the municipal waste (this is also the importance of the stated recycling symbol).
 - 2) Contact your municipal authority to indicate the point of disposal of your device for recycling.
 - 3) With the most useless your device in the right place
- Recycling helps to protect the environment and the operation again your device materials.

WARRANTY

The product has warranty for two (2) years in pump and six (6) months on the mattress and the other non-mechanical parts. The warranty covers factory defects and not damages resulting from improper use / maintenance of the product.

The warranty apply factory defects & DOES NOT apply damages resulting from improper use , poor maintenance, modification, abuse or non-compliance with the product's operating instructions.

Damage or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) and transportation companies are not covered by the warranty.

Warranty Form

Buyers Data

Name:

Date of purchase: S/N.....

LOT:..... Phone:

Resellers Data

Name :
.....
.....

Address:.....
.....

Phone:

Reseller Stamp

MOBIAM S.A
M.MPOTSARI 96-98
Chania-Crete, Greece
+30-28210-63222 www.mobiakcare.gr