EL EN

MOBIAK®

MEDICAL • GASES • FIREFIGHTING est. 1977

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PRODUCT DESCRIPTION

The rollator can be used by people with limited mobility and their weight must not get over 120 Kg
 The rollator has case for canes and crutches
 The rollator has break adjustment on the handler and on the rare wheels.

We reserve the right to make changes as it becomes necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased.

ASSEMBLY INSTRUCTIONS

- Push the seat until the rollator is fully opened and stabled.
- Adjust the height of the handlers and secure with the available knobs.
- Put the back rest in to the poles which are in front of the handlers by pushing the side buttons to secure.
- Place the basket in the front side of the rollator as shown in the photo.
- Secure the breaks on the frame of the rollator by using the safeties on it.

PRODUCT DIMENSIONS

DIMENSIONAL ITEMS	DIMENSIONAL MEASUREMENTS
DIMENSIONS OF SEAT (WIDTH x LENGTH)	35,5cm X 33cm
TOTAL WIDTH (REAR WHEELS)	52cm
TOTAL WIDTH (FRONT WHEELS)	63cm
SEAT HEIGHT (FROM THE GROUND)	53cm
HEIGHT RANGE FOR HANDLS(FROM THE GROUND)	79cm-95cm
MAXIMUM LOAD	120Kg
WEIGHT OF ROLLATOR	7,8kg

SAFETY ADVICE

What to notice before use

- When you stop to use the rollator you must rely on both handles equally. Support only on one hand (without escort) can damage the handle and may cause injury to the user
- When sitting or standing up from the rollator be careful your clothes not to be hooked in it.
- Regularly clean the rollator.
- Do not let the walker wet.
- Do not allow children near the walker without supervision
- Check periodically the rollator and if you find a damage please fix it at an authorized partner
- Do not use unauthorized parts other than those approved by Mobiak
- Don't put weight on the rollator over than the permissible.

IMPORTANT!

Make sure that the rollator which you use is fully open before the use

Cleaning instructions

- Use cold water and mild soap to clean the rollator
- Dry it with a soft cloth
- Dirt, oil or debris must be removed from the rollator

Maintenance

- Clean the rollator after each use with a soft cloth.
- Check every moving part if it moves fine and without problems
- Do a complete test - usage, before its use

DECLARATION OF CONFORMITY

We declare the conformity of the above product to the provisions of the Medical Equipment Directive 93/42/EEC and the Greek National Legislation ΔΥ8δ/Γ.Π.οικ.130648. ΦΕΚ 2198B/02-10-2009

Each product in this statement meets the technical specifications contained in its technical documentation If the appropriate maintenance will be provided and the product will be used according the “instruction of use” then the product is safe and effective and does not endanger the health safety of patients, user and or third parties.

This declaration is issued on the basis of a certificate of registration in the Manufacturer Register of Medical Devices of the Competent Authority (EOF). The product belongs in Class I, Classification rule 1 & Corresponds to the Medical Device Directive: 93/42/EEC

PROTECTING THE ENVIRONMENT

If you find one day that your product needs replacement, or no longer serves you, consider the environment

- 1) Do not dispose of your product along with the municipal waste (this is also the importance of the shown recycling symbol).
- 2) Contact your municipal authority to indicate the point of disposal of your device for recycling
- 3) You have to dispose of your useless product in the right place
- 4) Correct disposal of your appliance helps the protection of the environment as well as the recycling of the appliance components.

IMPORTANT

Do not use the product for a purpose not indicated in this manual

MOBIAC S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.

MOBIAC S.A reserves the right to change the information contained in this document without previous notice.

WARRANTY

The product has warranty for two (2) years from the date of sale of the product.

The warranty apply factory defects & **DOES NOT** apply damages resulting from improper use, poor maintenance, modification, abuse or non-compliance with the product's operating instructions. Also **DOES NOT** cover worn out parts such as tires, brakes, fabric and that can wear out during use or over time.

Damages or defects caused by: natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty.

Warranty Form

Buyers Data

Name:

Date of purchase:S/N.....

LOT:.....Phone:

Resellers Data

Name :
.....
.....

Address:.....
.....

Phone:

Reseller Stamp

MOBIAM S.A
M.MPOTSARI 96-98
Chania-Crete, Greece
+30-28210-63222 www.mobiakcare.gr