EL EN

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  
MEDICAL • GASES • FIREFIGHTING est. 1977

## PRODUCT DESCRIPTION

These wheelchairs have been designed for physically or mentally handicapped people with limited mobility only for transportation. This user manual is drafted from the latest specifications and product information available at the time of publication. MOBIAS S.A reserves the right to make changes as they regard it necessary without previous notice in order to improve its characteristics.

## PRODUCT DESCRIPTION

FRAME	Aluminum
COLOUR	Silver
SEAT and BACKREST	Fabric
ARMREST	Folding
FOOTREST	Detachable
REAR WHEEL (QUICK RELEASE)	Inflatable (24") / Solid (12")
FRONT WHEEL	Solid

## PRODUCT DIMENSIONS

BACKREST WIDTH	45 cm
SEAT WIDTH	46 cm
SEAT DEPTH	40 cm
BACKREST HEIGHT	45 cm
TOTAL WIDTH (REAR WHEELS)	50 cm
TOTAL LENGTH (WHEEL TO WHEEL)	80 cm
TOTAL LENGTH (INCLUDED FOOTRESTS)	105 cm
TOTAL HEIGHT	90 cm
HEIGHT (GROUND TO SEAT)	47 cm
WIDTH WHEEL TO WHEEL (FOLDED – REAR WHEELS)	30 cm
FRONT WHEELS	8"
REAR WHEELS	24" / 12"
MAXIMUM PATIENT'S WEIGHT	110 Kg
PRODUCT WEIGHT	19 Kg

## ASSEMBLY OF FOOTRESTS

### • FOR DISASSEMBLY

Push in, in order to free the security and turn the right footrest to the right. Follow the same procedure for the left footrest.

In order to remove the footrests pull them up from the wheelchair arm.

### • FOR ASSEMBLY

Install the footrests in the wheelchair putting the right footrest in the right side and the left footrest in the left side of wheelchair. Turn the footrests towards the center of the wheelchair until they stop, so they will automatically lock into the wheelchair. Reconnect the tibia strap.

## FOLDING ARMREST

### • FOR RELEASE

Push the security pin and pull the arm upwards.

### • FOR ASSEMBLY

Press the security pin down and lightly press the arm into its anchor points. Then release the security pin.

## ATTENTION !!!

- Make sure that the armrests have been locked before you use the wheelchair.
- Never lift the wheelchair from the armrests or footrests, as these components are removable and may cause damage to the wheelchair or injury to the user.

## WHEEL LOCKING

- Move the brake lever to the wheel if it is needed, for more braking action and remove the brake lever from the wheel for less braking action.
- Firmly tighten the lock nut.
- Check that the wheel spins freely when the lock is not applied, but is completely blocked when the lock is activated.

## INSTALLING / UNINSTALLING THE BACK WHEELS

1. Press the button which is in the middle of the wheel and pull out holding the wheel simultaneously.
2. Remove the pin from the wheel you removed
3. Repeat STEP 1 for the opposite rear wheel.
4. To replace the rear wheel follow the above steps (1-2) backwards.

### **Note:**

***Regarding the wheelchair ADAPT the rear wheels can be replaced with wheels of 12" if needed (which can be purchased with additional cost from an official MOBIAK dealer Code: 0811309)***

## **SAFETY ADVICE**

### **What to check before use**

- Quick-release axles on the rear wheels.
- Tires, tire pressure and wheel locks. The wheel locks will not work properly if the pressure is too low or the gap between the tire and the wheel locks is too long.
- The maximum user weight is 110 kg. This wheelchair should only be used to carry one person each time. Any other usage violates the security regulations.
- Users should not sit in the wheelchair while driving by any means of transport.
- When vehicles are in motion, unoccupied wheelchairs have to be secured with right tools. During driving, it is possible to cause damages to the wheelchair by the traps or uneven terrain.

### **PAY ATTENTION TO THE FOLLOWING**

- The construction of this wheelchair is designed to provide maximum safety.
- For your own safety, the following rules must be observed.
- Nonspecific or incorrect usage can increase the risk of accidents.
- Always wear light-colored clothes when moving in dark places so that you can be observed from others.
- Be careful during your usage in this wheelchair.
- Reflectors are available in the wheels and in the handlers of the wheelchair.
- **Do not** attempt to reach objects if you need to lift them from the floor reaching between your knees.
- **Do not** attempt to reach objects if you need to move on the seat.
- **Do not** use the wheelchair on motorways.
- **Do not** attempt to move on the surfaces with water, ice or oil membrane.
- **Do not** use unauthorized parts, accessories or adapters other than those approved by Mobiak S.A.
- **Do not** stand on the wheelchair.
- **Do not** use the armrest as a platform when entering or exiting the wheelchair.

## **WHILE DRIVING WITH YOUR WHEELCHAIR, DO NOT FORGET !!**

Always use lifts and ramps. If these are not available, you can reach your destination with the help of two escorts. They must catch the wheelchair by the safety points. A wheelchair should never be lifted by a passenger. It should be pushed only.

## **MAINTENANCE**

### **IMPORTANT**

Periodic review of the procedures below will help to maintain the wheelchair and the efficient function of it.

### **WEEKLY**

- Check the wheel pressure by following the instructions on the wheels.
- Check that the central bolt of the transverse arm is tight and in good condition.
- Clean the fabric with soapy water or mild detergent.
- Clean the metal parts of the chair with a dry cloth or with special products for chrome surfaces or painted surfaces.
- Check the wheel bolts and adjust them if it is necessary.
- Check the front and rear wheels for any damages.

### **QUARTERLY**

- Check wheel forks, pulling wheels, propellers and adjust them if it is necessary
- Grease all that components that are being subjected to friction or impending wear.
- Tighten the screws and nuts.
- General control of the chair's status and repair to whatever is necessary.

### **ANNUALLY**

We recommend a full check once a year to a qualified service provider.

In addition to maintenance and control, both the frame and the fabric can be thoroughly cleaned.

- To maintain its appearance, the fabric must be cleaned regularly.
- Stains can be removed with a sponge and a mild detergent with warm water, sweeping in the direction of texture.
- Rinse with clean water before letting it dry.
- Do not place the fabric in a tumble dryer. If the stain is deep, the surface should be cleaned with a soft brush.
- The frame should be cleaned regularly with a mild detergent.

## DECLARATION OF CONFORMITY

We declare the conformity of the above product to the provisions of the Medical Equipment Directive 93/42/EEC and the Greek National Legislation ΔΥ8δ/Γ.Π.οικ.130648. ΦΕΚ 2198B/02-10-2009

Each product in this statement meets the technical specifications contained in its technical documentation if the appropriate maintenance will be provided and the product will be used according to the "instruction of use" then the product is safe and effective and does not endanger the health safety of patients, user and or third parties.

This declaration is issued on the basis of a certificate of registration in the Manufacturer Register of Medical Devices of the Competent Authority (EOF). The product belongs in Class I, Classification rule 1 & Corresponds to the Medical Device Directive: 93/42/EEC

## IMPORTANT

Do not use the product for a purpose not indicated in this manual

MOBIAR S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.

MOBIAR S.A reserves the right to change the information contained in this document without previous notice.

## PROTECTING THE ENVIRONMENT

If your product needs replacement, or no longer serves you, consider the environment.

- 1) Do not dispose of your product along with the municipal waste (this is also the importance of the shown recycling symbol).
- 2) Contact your municipal authority to indicate the point of disposal of your device for recycling
- 3) You have to dispose of your useless product in the right place
- 4) Correct disposal of your appliance helps the protection of the environment as well as the recycling of the appliance components.
- 5) The electrical appliances due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards.

## WARRANTY

The product has warranty for two (2) years from the date of sale of the product.

The warranty apply factory defects & **DOES NOT** apply damages resulting from improper use, poor maintenance, modification, abuse or non-compliance with the product's operating instructions. Also **DOES NOT** cover worn out parts such as tires, brakes, fabric and that can wear out during use or over time.

Damages or defects caused by: natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty.

# Warranty Form

## Buyers Data

Name: .....

Date of purchase: .....S/N.....

LOT:.....Phone: .....

## Resellers Data

Name :  
.....  
.....

Address:.....  
.....

Phone: .....

**Reseller Stamp**

MOBIAK S.A  
M.MPOTSARI 96-98  
Chania-Crete, Greece  
+30-28210-63222 [www.mobiakcare.gr](http://www.mobiakcare.gr)