EL EN

MOBIAK[®]

MEDICAL • GASES • FIREFIGHTING est. 1977

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+30-28210-63222

www.mobiakcare.gr

**MOBIAK**[®]
MEDICAL • GASES • FIREFIGHTING est. 1977

CE

PRODUCT DESCRIPTION

Air cushions can reduce & cure bed ulcers in chronic diseased patients or for people who sit on wheelchair or armchairs for long periods. They can be used for home care use. They distribute evenly pressure on the body to prevent edema.

The air pump alternates the air pressure on the cushion with a feeling of light massage & activates immobilized cells due to the prolonged sitting.

This product is indicated for patients who spend more than 15 hours a day in a chair, armchair or wheelchair.

OPERATION USE

For air cushion:

- Check for sharp objects on the patient's seat (wheelchair, armchair, chair, etc.), and then place the air cushion on. Make sure the air inlet pipes are on the same side of the pump and are not folded.

- Attach the air cushion to the pump by using the hoses in the package and install correctly.

For the pump

- Use the hooks behind the pump to hang the pump onto the wheelchair armrest. Connect the pump to the nearest wall outlet (not an extension cord) and adjust the air flow according the weight of the patient with the rotary selector on the top of the device.

SPARE PARTS

REF. NUMBER	DESCRIPTION
0806297	STRIPED AIR CUSHION
0806272	PUMP FOR ANTI-DECUBITUS BUBBLE MATTRESS MINOA

DESCRIPTION OF INDICATORS AND FUNCTIONS FOR THE AIR PUMP

1. Power switch
2. Rotary pressure switch.
3. Air outlets

SPECIFICATIONS

PUMP	Voltage (V) - Frequency (Hz)	AC220-240V/ 50HZ
	Air Output	≥ 5-6 lit/min
	Pressure Range	50mmHg – 110mmHg
	Cycle Time	12min
	Power Consumption	≤ 7 W ± 10%
	Noise	≤ 30 dB ± 10%
	Dimensions (LxWxH)	22,5 x 10,5 x 8,1 cm
	Pump Weight	1 Kg
AIR CUSHION	Max. patient weight (Kg)	120 Kg
	Color	Blue
	Material	Nylon TPU & Stretch PU for the cover
	Cushion height (cm)	10 cm
	Dimension (LxWxH)	43 x 43 x 10 cm
	Air tubing length (cm)	75 cm
	Weight (Kg)	2 Kg

MAINTENANCE

To clean the cushion use a soft cloth or sponge & water with mild soap. **ATTENTION:**

- Do not use acidic & caustic dilutions.
- Do not smoke & do not use any flame near the cushion.
- Do not have the cushion in contact with direct heat sources.
- If you need to save the cushion, empty of air, by folding it
- Keep or store the pump in a clean, dry and well ventilated environment.
- Before you clean the pump, disconnect from the power supply.
- Clean the pump with a dry cloth.

ATTENTION:

Do not bring the pump in contact with liquids.

IMPORTANT (!)

CAUTIONS

- Do not disconnect the hoses while the pump is on.
- Do not use sharp objects near the cushion.
- Do not exceed the maximum load (kg) recommended by the manufacturer.
- Do not use, while bathing the patient.
- Protect the product by oils, lotions and creams, by covering it with the cotton wool fabric, or a bath towel.
- Do not use alcohol-based cleaning products to clean the cushion. This could damage the material of the layer.
- Clean the upholstery after each daily use.
We recommend to clean the upholstery with a mild detergent to protect it from oil stains and dust. A common practice is to use a spray bottle with a mixture of anti-bacterial soap and water. Spray the surface and then wipe with a clean towel. The best solution is to be cleaned with warm water mixed with a mild soap and then thoroughly dried.
- Protect the layer from scratches, cuts and punctures.
- Do not leave the product on, if it is not in a use.

STORAGE

- Switch off the device, unplug it from the power supply and disconnect the air-cushion from the pump.
- Place the air-cushion on a flat surface and roll or fold it from top to bottom.
- Allow air slowly escape the cushion to reduce storage space
- Close the protective cap cover on the quick connector.
- Place air-cushion & the pump in the carrying bag (if available) and store it in a moisture-free place.

IMPORTANT (!)

In case of damage or malfunction of the pump, please contact your authorized and trained agent.

CAUTION (!)

Do not use the product for a purpose not indicated in this manual

Only qualified person or trained for the treatment and prevention of bedsores, can operate with the device

MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the product.

MOBIAK S.A reserves the right to change the information contained in this document without previous notice.

EC Declaration of Conformity

We declare the conformity of the above product to the provisions of the Medical Equipment Directive 93/42/EEC and the Greek National Legislation ΔΥ8δ/Γ.Π.οικ.130648. ΦΕΚ 2198B/02-10-2009

Each product in this statement meets the technical specifications contained in its technical documentation if the appropriate maintenance will be provided and the product will be used according to the "instruction of use" then the product is safe and effective and does not endanger the health safety of patient's, user and or third parties

This declaration is issued on the basis of a certificate of registration in the Manufacturer Register of Medical Devices of the Competent Authority (EOF).

The product belongs in Class I, Classification rule 1 & Corresponds to the Medical Device Directive: 93/42/EEC

Environmental Protection

If your device needs replacement ,or you do not use it, consider the environment protection:

- 1) Do not dispose your appliance with other waste (this the meaning of the recycled symbol).
- 2) Contact your local authority to instruct you the recycling centers for your appliance.
- 3) Recycling your no use appliance, you protect the environment and help the exploitation of new materials from your device .

WARRANTY

The product has warranty for two (2) years in pump, six (6) months on the cushion. The warranty covers factory defects and not damages resulting from improper use / maintenance of the product. The warranty apply factory defects & DOES NOT apply damages resulting from improper use , poor maintenance, modification, abuse or non-compliance with the product's operating instructions. Damage or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) and transportation companies are not covered by the warranty.

Warranty Form

Buyers Data

Name:

Date of purchase:S/N.....

LOT:.....Phone:

Resellers Data

Name :

.....

Address:.....

Phone:

Reseller Stamp

MOBIAK S.A
M.MPOTSARI 96-98
Chania-Crete, Greece
+30-28210-63222 www.mobiakcare.gr

MOBIAK[®]
MEDICAL • GASES • FIREFIGHTING est. 1977