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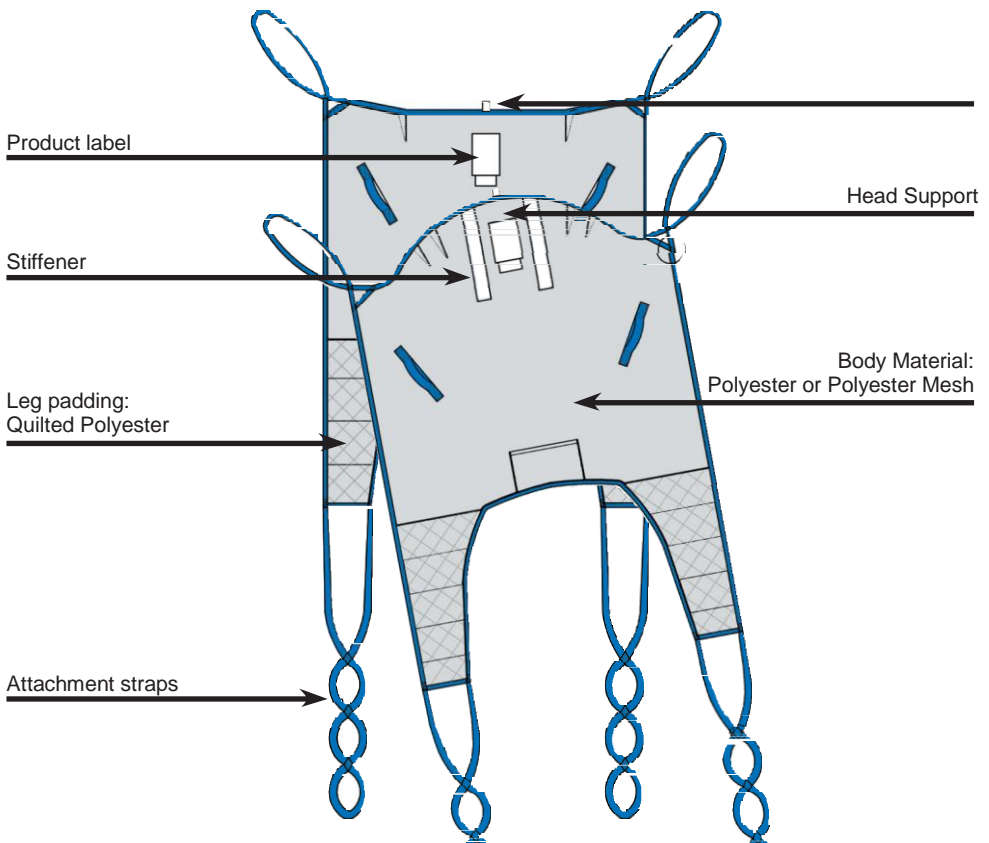
Washing instructions:

Warning

Always read this instruction and the instruction for the lifter to be used with the sling

Technical Specifications

Code	Description	Max. Weight	Material
0803121	Universal Sling HS, Loop Straps attachment	275kg	Polyester

Definition in this IFU

Warning: Means Safety warning. Failure to understand or obey this warning may result in injury to you or to others.

Note: Means important information for the correct use of this system or equipment.

Intended Use

The Universal Sling HS (with Head Support) is designed with split legs and full shaped body for users who requires head support. Suitable for most transfers in a seating or laying position and where user have little neck and/or head control. The Universal Sling will assist the caregiver for lifting and transfer a patient in either a sitting or laying position, from and to the bed and/or wheelchair.

Safety instructions

- Always check the label is readable and that the sling is in good condition without signs of wear and tear.
- Always make sure not to exceed the Max. Weight defined for the product.
- Always read the instruction manual for the lifting equipment to be used with the sling.

Before and after every use

1. Always perform a risk assessment before starting the specific lifting session in order to make sure the product model is correct and that the lifting procedure is fully understood.
2. Before using the sling, read this instruction manual and the instruction manual for the lifting equipment to be used
3. When soiled or stained the sling must be washed before use.
4. **Warning:** Check the sling for any fraying, loose stitching, attachments or any deteriorations.
5. After use, check sling according to the step 3 and 4 above and store in a dry area.

Service & Maintenance

The sling shall be checked by authorized personnel every 6 months according to ISO10535

Customer support/Trouble shooting

- Please refer to the manual for the lifting equipment used
- In doubt please contact your dealer who supplied the product

Expected service life & Disposal

Expected Service Life varies between 1–5 years, depending on usage and number of washing cycles. When washed once a week the sling has an Expected Service Life of 2 years. Expected Service Life is based upon a correct usage and washings procedure in accordance with this IFU. Dispose the slings according to the local regulations.

Storing and Shelf life

The sling should be stored in a dry and clean environment. The shelf life is 3 years in original packaging. Store in temperatures between 0°-40° C (+32°-104°F)

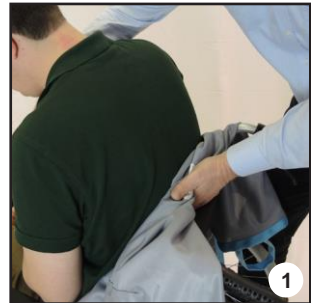
Product care & Maintenance

- Always follow the instructions on the product label and check the sling for deteriorations.
- Mark the sling with the specific patient name and keep in dry and clean environment.


Applying the Universal Sling in Chair

1. Lean the user slightly forward and position the Sling behind the user's back and head. The label side facing outward and the center line of the sling aligned with the user's spine.

Place the hand in the position pocket and carefully push the sling as far down the user's lower back as possible. (see pic 1)

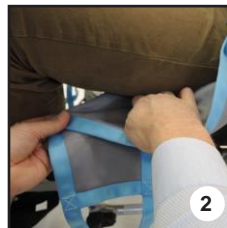Place the sling legs alongside the user's legs. Fold the sling leg (see pic 2) and carefully feed it underneath the user's legs.


After positioning the sling legs (see pic 3,4) cross the leg straps by putting one through the other. (see pic 5)

Recline the user into the sling, ensuring both arms are contained within the sling.

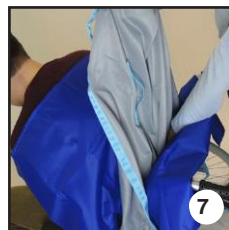Note! If required, two flat slide sheets can be used to apply the sling.
(See pic 6,7,8,9)

2. Position the hanger bar carefully above the user.
(See pic 10).

3. Place the four loop straps on the hanger bar attachment hooks and pull the straps down until locked in position.
(See pic 11,12)

Note: Always check the lifter instruction manual for details

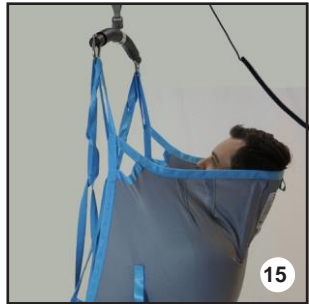
4. Check user position and carefully make the required adjustments to achieve the best comfort for the user.

Check the support of the head and that arms and legs are in the right positions. (See pic 13)

5. Raise the hanger bar until all straps start to tighten and become tense enabling you to check that **ALL** the straps are securely attached. (See pic 14, 15)

6. Carefully perform the lift, whilst remaining vigilant of the user's position and comfort. (See pic 16)

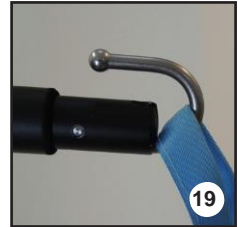
7. Once the process of transfer is complete, return the user to a chair or bed and slowly lower the hanger bar until all straps are loose. (See pic 17)

8. Release the loop straps from the hanger bar by pulling the strap upwards. (See pic 18,19)


Note: Always check the lifter instruction manual for details



Applying the Universal Sling in Laying position

1. Roll the user to the side, facing away from you.

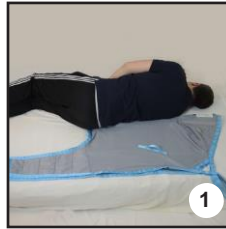Fit the sling by folding it on the middle. The label side facing outwards. Then place the sling lengthwise with the users back. (See picture 1)

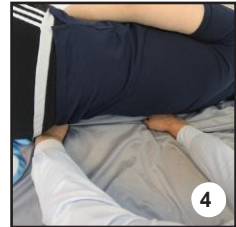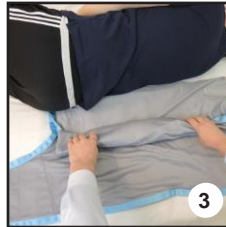Roll up top half of the sling and gently tuck it under the user. (See picture 2,3,4)


2. Roll user to the other side, now facing you. Gently pull the top half of the sling through, when fully unraveled flatten it. (See picture 5)



Roll the user to a backside position. (See picture 6)

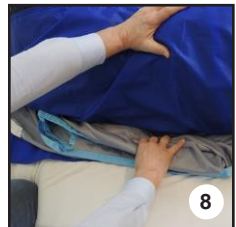
Note! If required, two flat slide sheets can be used to apply the sling. (See picture 7,8)

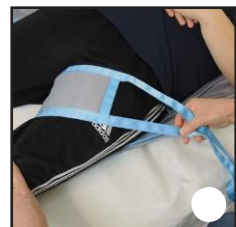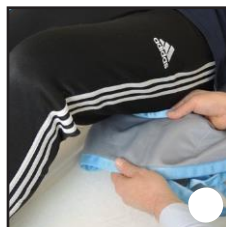3. Bend the users leg closest to you and insert sling leg under the users thigh. (See picture 9,10)

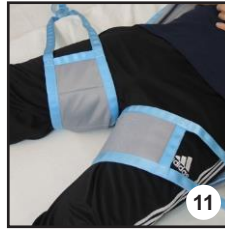Unbend the knee and redo procedure on the users opposite leg.
(See picture 11)

After positioning the sling legs properly, cross the leg straps by putting one through the other.
(See picture 12)

4. Position the hanger bar carefully above the user.
(See picture 13,14)

5. Place the four loops straps on the hanger bar attachment hooks and pull the straps down until locked in position.
(See picture 15,16)

Note: Always check the lifter instruction manual details.

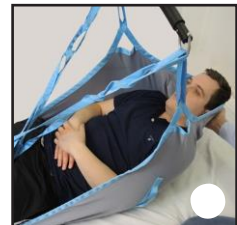
6. Check user position and carefully make the required adjustments to achieve the best comfort for the user. Check the support of the head and that arms and legs are in the right positions.
(See picture 17)

Note: If required, recline the bed so the user is in a half-sitting position.

7. Raise the hanger bar until all straps start to tighten and become tense, enabling you to check that **ALL** the straps are securely attached.
(See picture 18)

8. Carefully perform the lift, whilst remaining vigilant of the user's position and comfort.

Remember to support the users head when raising from the bed.
(See picture 19,20)

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
9. Once the process of transfer is complete, return the user to a chair or bed and slowly lower the hanger bar until all straps are loose.
(See picture 21)

10. Release the loop strap from the hanger bar by pulling the strap upwards.
(See picture 22,23)

Note: Always check the lifter instruction manual details.

EC Declaration of Conformity

We declare the conformity of the above products to the provisions of the Medical Equipment Directive 93/42/EEC and the Greek National ΔΥ86/Γ.Π.οικ.130648. ΦΕΚ 2198Β/02-10-2009

This declaration is issued on the basis of a certificate of registration in the Manufacturer Register of Medical Devices of the Competent Authority

The product belongs in Class I, Classification rule 1 & Corresponds to the Medical Device Directive: 93/42/EEC

Each product in this statement meets the technical specifications contained in its technical documentation. If the appropriate maintenance will be provided and the product will be used according to the "instruction of use" then the product is safe and effective and does not endanger the health safety of patient's, user and or third parties.

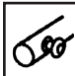
ENVIRONMENTAL PROTECTION

If your product needs replacement, or no longer serves you, consider the environment:

- 1) Do not dispose of the product along with the municipal waste (this is also the importance of the stated recycling symbol).
- 2) Contact your municipal authority to indicate the point of disposal of your product for recycling.
- 3) Recycling helps to protect the environment and the operation again your device materials.

Description of label symbols. Applicable symbols printed on product label

	<p>Patient Name Le nom du patient Nombre del paciente Patientenname Nome del paziente Naam van de patiënt 患者姓名</p>		<p>Date of manufacturer /month Date de fabrication / mois Fecha de fabricación / mes Herstellungsdatum / Monat Data di produzione / mese Fabricagedatum / maand 製造日期</p>		<p>Do Not Bleach Ne pas javelliser No usa blanqueador Nicht bleichen Non candeggiare Niet bleken 不能漂白。</p>
	<p>Manufacturer Fabricant Fabricante Hersteller Fabbricante Fabrikant 生產廠家</p>		<p>Read the instruction for use Lire les instructions d'utilisation Lea las instrucciones de uso Lesen Sie die Gebrauchsanweisung Leggere le istruzioni per l'uso Lees de instructies voor gebruik 請閱讀使用說明書</p>		<p>Do Not Tumble Dry Ne pas sécher par culbutage No utilizar secadora Nicht im Wäschetrockner trocknen Non asciugare Niet in de wasdroger 不可烘乾</p>
	<p>Article number Numéro de l'article Número del artículo Artikel-Nummer Numero di articolo Artikelnummer 貨號</p>		<p>EU representative EU Représentant EU Representante EU Vertreter EU Rappresentante EU Vertegenwoordiger EU 代表</p>		<p>Tumble Dry at Low Heat Sécher par culbutage à basse température Secar a baja Heat Wäschetrockner trocknen bei niedriger Hitze Nell'asciugatrice a bassa calore Drogen op lage temperatuur 低温滚筒烘干</p>
	<p>Lot Serial number Numéro de série de LOT Número de serie LOT LOT Seriennummer Numero di serie LOT Lot serienummer 批号</p>		<p>Declaration of compliance Déclaration de conformité Declaración de cumplimiento Konformitätserklärung Dichiarazione di conformità Verklaring van conformiteit 符合性聲明</p>		<p>Do Not Iron Ne pas repasser No planchar Nicht bügeln Non stirare Niet strijken 不可熨燙</p>
	<p>Serial Number Numéro de série Número de serie Volgnummer Numero di serie Volgnummer 序列号</p>		<p>Attachment by straps with loops Fixation par sangles avec boucles Fijación por correas con bucles Befestigung durch Gurte mit Schlaufen Fissaggio con cinghie con anelli Bevestiging door bandjes met lussen 通过挂带连接</p>		<p>Do Not Dry Clean Ne pas nettoyer à sec No lavar en seco Nicht chemisch reinigen Non lavare a secco Nicht chemisch reinigen 不可乾洗</p>
	<p>Material Symbol Symbole du matériel Símbolo material Materialen symbol Simbolo material Materiaal symbol 材質符號</p>		<p>Attachment by straps with Clips Fixation par sangles avec clips Fijación por correas con clips Befestigung durch Gurte mit clips Fissaggio con cinghie con clips Bevestiging door bandjes met clips 通过扣子连接</p>		<p>Recommended temperature for washing La température recommandée pour le lavage La temperatura recomendada para el lavado Empfohlene Temperatur zum Waschen Temperatura consigliata per il lavaggio Aanbevolen temperatuur voor het wassen 推薦使用溫度洗滌</p>
	<p>Service label Étiquette de service Servicio de etiqueta Service-Etikett Etichetta di servizio Dienst label 服務標籤</p>		<p>Safe Working Load Capacité de levage maximum Carga de trabajo segura Maximale hebelast Carico massimo di sollevamento Maximale tilcapacitet 安全工作載荷</p>		<p>Do not wash Ne pas laver No Lave Non lavare Nicht waschen Niet wassen 不可水洗</p>
			<p>Medical Device Dispositif medical Producto sanitario Medizinprodukt Dispositivo medico Medisch hulpmiddel 医疗器械</p>		<p>Do not use. Ne pas utiliser</p>

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