**GR EN**

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[www.mobiakcare.com](http://www.mobiakcare.com)

## INTENDED USE

The wheelchair is used for transporting people with disabilities or limited mobility on the beach.

The wheelchair provides comfort in transport due to its construction and at the same time contributes to the stimulation of psychology as it provides all kinds of facilities for people with disabilities or limited mobility.

## SPECIFICATIONS

FRAME	Aluminum
COLOR	Black
REAR WHEELS	Compact
FRONT WHEELS	Compact

## DIMENSIONS

											
0805310	49cm	49cm	42cm	80cm	93cm	137cm	134cm	31cm	17" 11"	120kg	12,5kg

## ASSEMBLY

Please refer on **ANNEX A**

## SYMBOLS

Please refer on **ANNEX B**

## MAINTENANCE

Periodic review of the procedures below will help to maintain the life and effectiveness of your wheelchair.

### ➤ BEFORE USE

- Check retaining fasteners connecting the individual segments.
- Check the wheelchair's front and rear wheels for possible damage, as well as in the fabric of the seat and the back.

### ➤ AFTER EACH USE

- Clean the fabric with a soapy waterway or mild cleanser.
- Clean the metal parts of the wheel with a dry cloth or with special products for painted surfaces.

### ➤ ANNUALLY

We recommend a complete review once a year in a specialized service provider.

- Lubricate with grease or oil all components subject to friction or wearing.
- Tighten the screws and nuts again.
- General revision of the stream of the wheel and repair everything that is necessary.

## CLEANING

In addition to maintenance and control, general cleaning can be done both the frame and fabric.

- To keep his appearance, fabric must be cleaned regularly.
- Dirt can be removed with a sponge and a mild detergent diluted in hot water, wiping in the direction of texture.
- Rinse with clean water before leaving dry.
- If the dirt is deep, the surface should be cleaned with a soft brush.
- The frame must be cleaned regularly with a mild cleanser.

## IMPORTANT!

The design of this wheelchair is developed to provide maximum safety. Nevertheless, users may be at risk using their wheelchairs inappropriately. For your own safety, the MANUAL rules must be followed. Non-specialized or incorrect adaptation or adjustment work can increase the risk of accidents.

### **WARNING**

In case of problem, do not use the wheelchair until the problem is solved.

- Do not use the product for a purpose not indicated in this manual.
- The manufacturer declines all responsibilities for any consequences resulting from an incorrect use of this product.
- The manufacturer reserves the right to change the information contained in this document without notice.
- Do not allow children to play on the wheelchair.
- Do not attempt to lift the wheelchair by any removable parts.
- Keep the packed wheelchair away from heat sources.

## **ENVIRONMENTAL PROTECTION**

Should you find one day that your product needs to be replaced or if it is of no further use to you, think of the protection of the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product components.

## WARRANTY:

Your wheelchair is guaranteed for 2 years.

The warranty covers factory defects & does NOT cover defects resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product instructions for use. It also does NOT cover worn parts such as brakes, belts, straps, wheel tires and whatever it is going to be damaged during use or over time. Damage or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where provided), transport companies are also not covered by the warranty. Shipping costs for goods of warranty are to be paid by the customer or by dealer.

## WARRANTY CLAIM FORM

BUYERS DETAILS			
FULL NAME:			
PURCHASING DATE:		PHONE:	
SELLER DETAILS			
FULL NAME		ADDRESS	
PHONE		<b>SIGN AND SIGNATURE</b>	

## **E.C. Declaration of Conformity**

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

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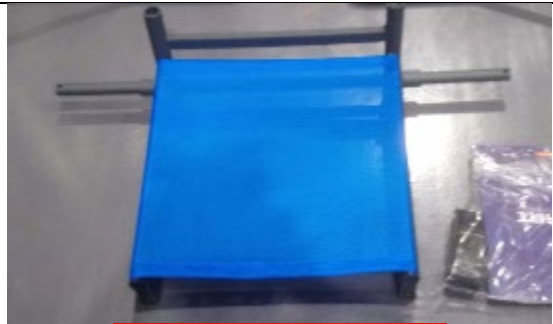
Place the wheels on the seat and fasten them with the retainers.

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Place the security belt and then the backseat and the retainers.

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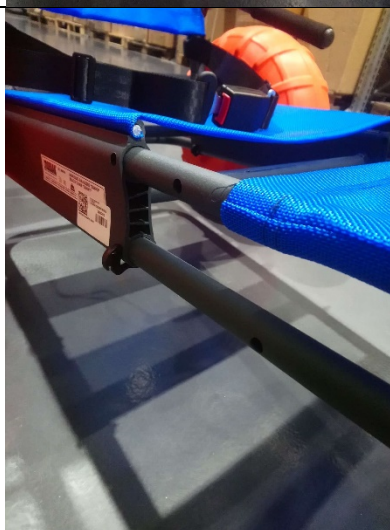
Place the foot support bracket and the wheel in the corresponding positions.

Place the retainers according to the patient's requirements.

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DESCRIPTION OF SYMBOLS THAT HAVE BEEN PRINTED ON THE LABEL  
MASTER CARTON AND OR USER MANUAL

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MOBIAK S.A